



Features & Benefits:

- Quickly and easily divide ramp into shorter, lighter sections.
- Easy Push button mechanism inserts and secures the pin.
- Replaces bolts at any location where two ramp links connect.
- Easily allows length to be adapted quickly without tools.



How To Use Quick-Connect Pins:

1. Align the two ramp sections so the bolt holes line up.
2. Push button on Quick Connect Pin and insert into holes.
3. Ensure the pin goes through both ramp sections.
4. Repeat for opposite side in same manner.

Scan QR code to watch a video tutorial:

