


# SUPPORT STANDS



Supports stands come as a pair and are attached to the sides of the ramp.

## Stand Installation for Roll-A-Ramp®:

1. Position stands at approximately the center of the ramp or every 6-8 ft. for longer lengths of ramp. Listed adjustment range of support stands are measured from the ground to the tread surface on the ramp where the stand will be mounted. Be sure stands are installed on the same tread. Line up the mounting hole in stand top with lower slot on the desired ramp tread. Position the stand in the middle of the slot on the ramp tread.
2. Hold the T-Nut inside of the ramp tread where the stand will be mounted. Then insert the T-Handle bolt (or Hex Head Bolt) with washer from outside through the stand top and into the T-Nut and tighten firmly.
3. Adjust the stand height by spinning the adjustment handle on the base of the support stands so the ramp is supported, but not lifting or creating a high point on the ramp.

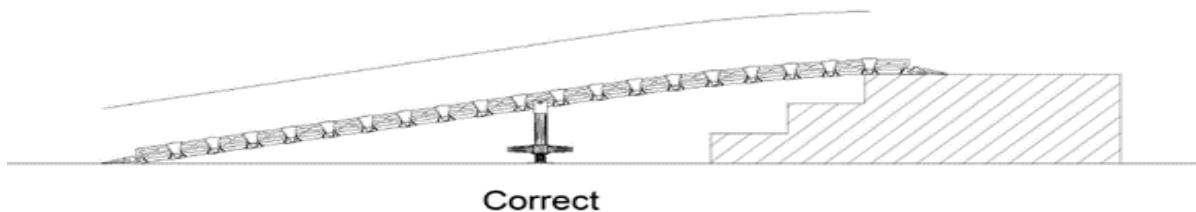

For a Visual Demonstration  
Watch Roll-A-Ramp How to Add Support Stands:  
YouTube: <https://tinyurl.com/How-to-add-stands>



### Support Stands (Sold in Pairs)

Stands are required for ramps 10' and longer

Position stands close to center, or approximately every 6—8 foot increment on longer ramps. Height is measured from ground to tread surface.

X Short	6-1/2" - 11"
Short	10-5/8" - 15-1/4"
Long	16-1/2" - 24-7/8"
X Long	24-1/2" - 38-7/8"
2X Long	36-1/8" - 52-3/4"