

HOW TO ASSEMBLE YOUR



Bath Bench with Back and Arms

Item # 9026
Capacity not to exceed 300 lb



No tools required.
Quick Assembly.

Follow these
SIMPLE STEPS



Remove all parts from box:

- One seat with legs and backrest tubing attached
- Backrest
- 2 arm rests



Place shower bench upside down on a level surface, aluminum components facing up. Adjust height of legs by depressing the push button and slide legs up to desired height. Push button must “click” into numbered hole. Legs must be the same height.



Turn seat on side with tubing holes facing you. Push seat back tubes into the base tubing until you hear a “click” sound as brass button goes through hole in tubing, making sure the button is completely through the hole.



Turn seat up right, grab tube arms and depress brass button, push firmly into openings. Button must be completely through the hole.

HOW TO USE

Use the Bath Seat when support is required in bath or shower. Arms may be used to lower or raise oneself up from and into the chair. Use the arms for stability and balance when needed.

CARE AND MAINTENANCE

Check regularly to make sure the all brass buttons are securely locked in holes and that the bath seat is stable before sitting. Use a mild detergent and warm water to clean bath seat.

WARNING

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. **NOVA shall have no responsibility for failures resulting from improper assembly or use.** Do not misuse the product.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts (rubber tips, plastic or metal hardware)	One Year Limited Warranty

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Regular cleaning is recommended. Use a non-abrasive cleaner or detergent, rinse with warm water and dry completely. Abrasive cloths may damage the surface. Regularly inspect your item and it's parts for any corrosion, breakage, compression or wear.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

- Chair legs must be adjusted to same height (number) so the bath seat sits level
- Make sure legs and backrest are secure (brass button locked completely into hole) before using
- Check before each use
- Use on a level surface
- DO NOT stand on Bath Seat

Questions?
Please contact us at
1-800-557-6682
info@novajoy.com

Thank you for choosing NOVA!

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE

Connect with us!
novajoy.com
facebook: novamedicalprod
twitter: novamedicalprod
twitter: novawalkerstalk
YouTube: NOVA Medical Products