

HOW TO INSTALL YOUR

CONTACT US!
We're here to help



nova

CANE HOLDER FOR STAR SERIES

Item # CH-4200R




info@novajoy.com



1 800 557 6682

TOOLS NEEDED

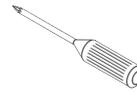

Phillips Screwdriver

**Quick installation
with step-by-step
instructions**

WHAT'S INCLUDED

a. (2) Mount arms



a.

b. (2) Mounting brackets



b.

c. (4) Phillip's screws



c.



1 Determine on which side, left or right, the cane holder will be installed. The rear legs, above the seat platform, are suggested areas.

2 Snap 1 of the mount arms onto outside of frame. Hook should face towards the rear of the walker. Snap the other arm onto the frame 2 inches below the first. Hook should face towards the front of the walker.



3 Attach 1 mounting bracket with 2 screws to upper arm and tighten. Repeat for lower arm.



4 To attach cane, position tubing in between the arms, facing the openings of the hooks. Push cane forward until it engages into the cane holder.