

Quick Start Guide


GA541 – Avenger



Read before assembly.


Step 1.

Use one hand to grasp a handgrip, and use your other hand to loosen the adjustment lever. Position the tiller to a comfortable driving position, tighten the adjustment lever to secure the tiller. See figure 1.

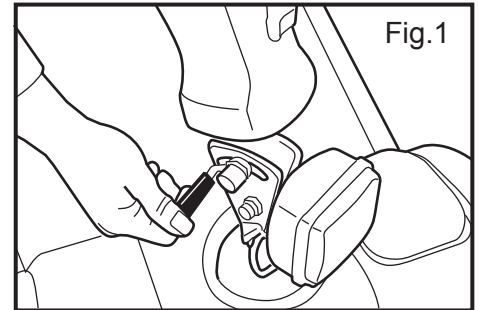


Step 2.

Remove the battery shroud. See figure 2.

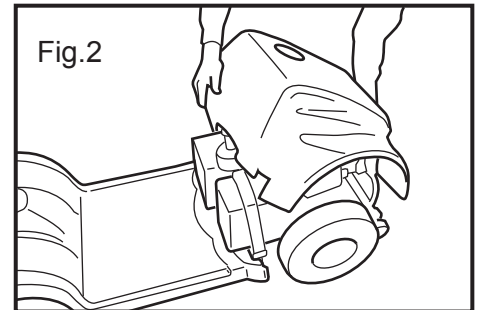

Step 3.

Carefully lower each battery into the battery tray on the scooter. Make sure that each battery is secured with the hoop and loop straps. Plug each battery into the scooter making sure the connectors are secured. Make sure the freewheel lever is in the engaged position. See figure 3.

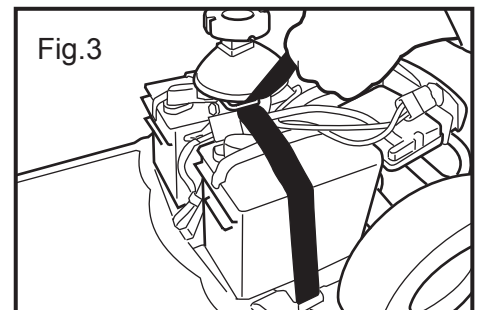


Step 4.

Hold the seat at an angle to visually align the seat swivel post, and the seat post on the scooter. Lower the seat swivel post into the seat post. Push down to make sure it is seated securely and unable to move side to side. Pull up on the swivel handle and move the seat so that it is facing in the forward position if not already there.

Step 5.

Insert each arm into the arm receiver tube at the bottom rear of the seat frame. Adjust the width of each arm. Using the supplied Allen wrench, install and tighten each arm width adjustment bolt to secure each arm in place. See figure 4.

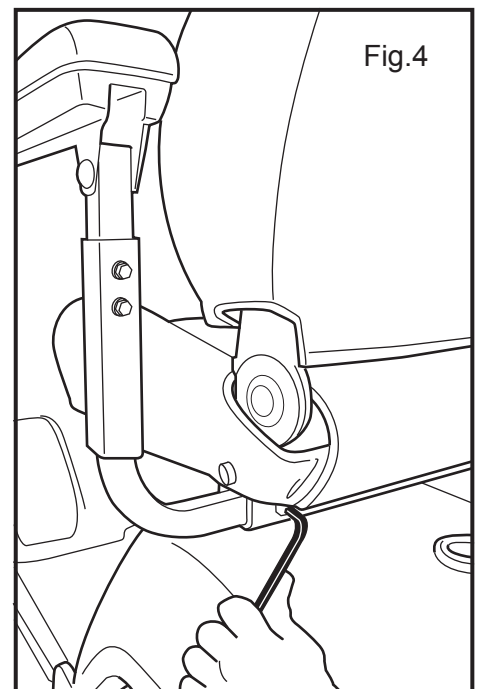

Step 6.

Sit on the seat, facing forward, insert key and turn the key to the right. Verify that the battery meter on the dash illuminates indicating power on and that there are no LED segments flashing. If there are LED segments flashing, count the number of segments, and refer to your owner's manual to determine the problem. If the battery meter is illuminated and there are no segments flashing, your scooter is assembled correctly, and you are ready to ride.

NOTICE

New batteries must be fully charged prior to the initial use of the product. Connect the charger to the scooter first, then the wall outlet, and charge the batteries for 12 hours before using the product.