

Quick Start Guide


GP301 – Stride



Read before assembly.


Step 1.

Connect the battery harness located behind the battery pack. See figure 1.

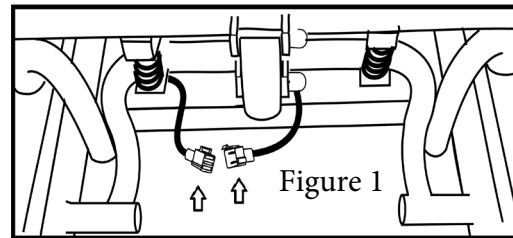

Step 2.


Align the battery pack with the rail under the seat. While squeezing the battery pack latch, slide it forward until it stops. Release the latch to lock the battery pack. See figure 2.

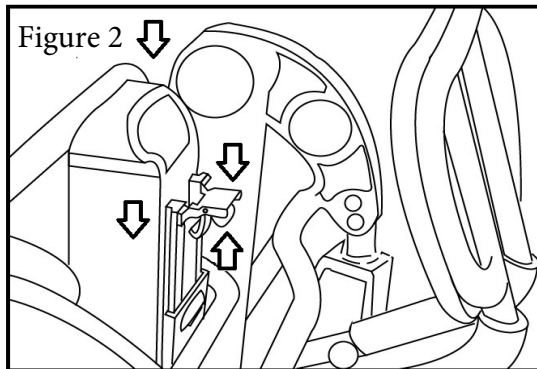

NOTICE New batteries must be fully charged prior to the initial use of the product. After completing setup, connect the charger to the power chair first, at the joystick or at the battery pack, then the wall outlet, and charge the batteries for 12 hours before using the product. See figure 7.


Step 3.

With one hand on the backrest push handle and the other hand on the seat tube, pull in opposite directions to unfold the power wheelchair. Note: An audible “click” indicates the latch has engaged and is now in the locked position. See figure 3.

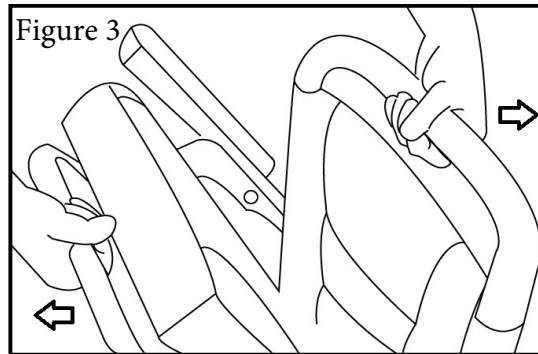


Step 4.

Loosen the armrest knob, insert the joystick on either arm then tighten the knob to secure the joystick. See figure 4.

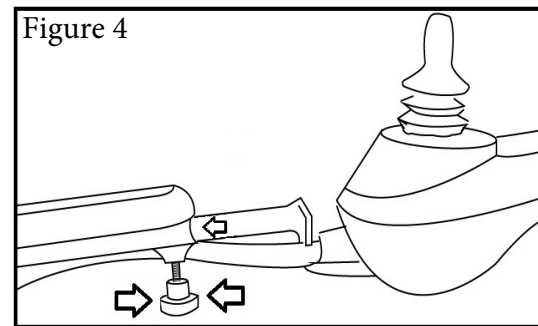

Step 5. Press the latch for one of the anti-tip brackets, which are located at the inside of the rear wheels under the motors. Fully extend the anti-tip until it locks into place. See figure 5. Repeat step 5 for the opposite anti-tip.

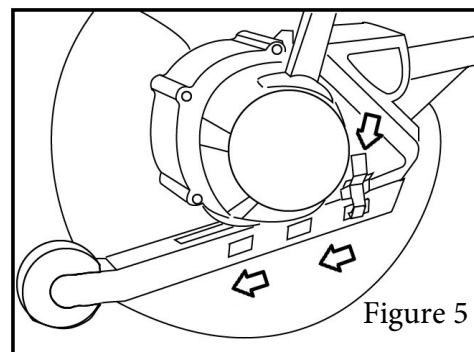

Step 6.

Place the two freewheel levers into the drive position. Note: The power wheelchair will not drive in freewheel. See figure 6

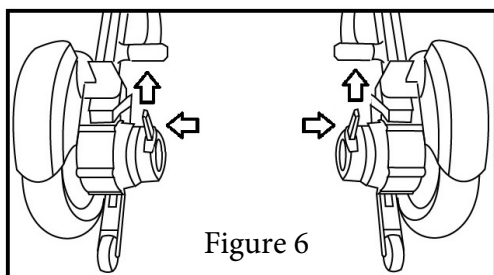


Figure 7

